

How to Connect Your Scanner to TPS Digital Dental Lab



ITERO (ALIGN)

CALL ITERO AT 1-800-577-8767 AND REQUEST TO ADD **TPS VENEERS (210116)** TO YOUR "FAVORITE LAB" LIST.
ON THE LAB LIST, SEARCH FOR **TPS VENEERS**. SELECT OUR LABORATORY AS A "FAVORITE LAB".
SEND YOUR SCAN: WE WILL RECEIVE THE SCAN AND BEGIN PROCESSING YOUR CASE.

3SHAPE TRIOS

LOG INTO 3SHAPE COMMUNICATE
GO TO CONNECTIONS → ADD A CONNECTION
SEARCH BY EMAIL: **SCANS@TPSDIGITALLAB.COM**
(WE'LL ACCEPT THE INVITE ON OUR END)

MEDIT

LOG IN TO YOUR MEDIT LINK ACCOUNT.
SELECT "ALL ORGANIZATIONS FROM THE SIDEBAR."
SEARCH FOR "**TPS DIGITAL DENTAL LAB**" AND CLICK 'REQUEST CONTRACT'.

PLANMECA

OPEN YOUR PLANMECA ROMEXIS SOFTWARE
SELECT A CASE AND CLICK "CLOUD EXPORT"
SELECT TPS DIGITAL DENTAL LAB BY ENTERING THIS EMAIL ADDRESS:
SCANS@TPSDIGITALLAB.COM
INCLUDE ALL NECESSARY ATTACHMENTS

SIRONA PRIMESCAN / OMNICAM (DS CORE)

LOG INTO DS CORE (REPLACES CEREC CONNECT)
GO TO PREFERRED LABS → ADD PREFERRED LAB
TYPE IN: TPS VENEERS (**TPSVENEERS@OUTLOOK.COM**)

DEXIS

LOGIN TO DEXIS IS CONNECT AS A DENTIST.
IN THE TOOLBAR, CLICK PARTNERS
IN THE PARTNERS WINDOW, CLICK INVITE A PARTNER.
IN THE EMAIL FIELD, ENTER: **TPSVENEERS@OUTLOOK.COM**
IN THE MESSAGE BOX, EITHER LEAVE THE DEFAULT TEXT OR WRITE YOUR OWN MESSAGE.
CLICK SEND. THE INVITATION MESSAGE IS SENT TO TPS DIGITAL DENTAL LAB AND A PARTNER ENTRY IS CREATED IN THE LIST OF PARTNERS

QUESTIONS? CALL OR TEXT 502-495-6406